

FLASH Piccolo™

LED Color Selection Guide

Each FLASH VetID™ Test Cartridge is designed to run at ideal reaction conditions, indicated by a color mark on the cartridge's reaction tube. Before inserting the cartridge, select the matching LED color on the FLASH Piccolo™ so it runs the test at the optimal temperature conditions.

- 1 Locate the color mark on your Test Cartridge's reaction tube
- 2 On the FLASH Piccolo™, press the button repeatedly to cycle through the available LED colors
- 3 Stop when the selected LED color matches the color mark on your cartridge
- 4 Insert the Test Cartridge to begin the test

Product Name	Cat No.	LED Color
FLASH VetID™ FHV	VID-01-01	● ● ●
FLASH VetID™ FCV	VID-01-02	● ● ●
FLASH VetID™ B. bronchiseptica	VID-01-03	● ● ●
FLASH VetID™ C. felis	VID-01-04	● ● ●
FLASH VetID™ M. felis	VID-01-05	● ● ●
FLASH VetID™ FPV	VID-01-06	● ● ●
FLASH VetID™ CPV	VID-01-07	● ● ●
FLASH VetID™ CDV	VID-01-08	● ● ●
FLASH VetID™ B. gibsoni	VID-01-09	○ ○ ○
FLASH VetID™ E. canis	VID-01-10	● ● ●
FLASH VetID™ Lepto	VID-01-11	● ● ●
FLASH VetID™ B. vogeli	VID-01-12	● ● ●
FLASH VetID™ B. canis	VID-01-13	● ● ●
FLASH VetID™ L. infantum	VID-01-14	● ● ●
FLASH VetID™ A. platys	VID-01-15	● ● ●
FLASH VetID™ FeLV	VID-01-16	● ● ●
FLASH VetID™ FIV	VID-01-17	● ● ●
FLASH VetID™ APV	VID-01-18	● ● ●
FLASH VetID™ BFDV	VID-01-19	● ● ●
FLASH VetID™ C. psittaci	VID-01-20	● ● ●

Important: Selecting the wrong LED color will run the test at the wrong temperature and may produce an invalid or inaccurate result. Always confirm the color match before inserting the cartridge.